

ChloroPrep® 3 ml cutaneous solution

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v^{1,2}
Single patient use, sterile, non-touch, skin antisepsis system

For procedures such as:

Central venous catheter insertions, insertion site maintenance,
midline catheter insertions, minor procedures

Coverage area 15 cm x 15 cm



Pinch

Hold the applicator as shown, being careful not to touch the sponge.

Pinch the wings to release the solution. You will hear a 'pop' as the ampoule breaks.



Apply

Starting at the incision site, gently press the applicator against the skin and apply using repeated up and down, back and forth strokes.

Apply to this area for at least 30 seconds before working outward towards the periphery.



Dry

Leave for approximately 30 seconds, allowing the area to air dry **completely** before applying sterile drape.

Discard the applicator after a single use.