

ChloroPrep® FREPP 1.5 ml cutaneous solution

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v^{1,2}
Single patient use, sterile, non-touch, skin antiseptics system

For procedures such as:

Blood culture collections, peripheral canulation, AV fistula
and 'portacath' access, simple biopsies and minor procedures

Coverage area 10 cm x 13 cm



Pinch

Hold the applicator as shown, being careful not to touch the sponge.

Pinch the wings together. You will hear a 'pop' as the ampoule breaks.



Apply

Gently press the applicator against the skin and apply the antiseptic using repeated up and down, back and forth strokes for about 30 seconds.



Dry

Leave for approximately 30 seconds, allowing the area to air dry **completely** before proceeding.

Discard the applicator after a single use.