

## ChloroPrep® SEPP .67 ml cutaneous solution

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v<sup>1,2</sup>

Single patient use, sterile, non-touch, skin antiseptics system

### For procedures such as:

Peripheral cannulation, surgical pin site cleansing,  
subcutaneous device insertions

Coverage area 5 cm x 8 cm



### Pinch



Hold the applicator as shown, being careful not to touch the tip.

Pinch the sides of the applicator until you feel the inner ampoule break.



### Apply



Gently press the applicator against the skin and apply the antiseptic using repeated up and down, back and forth strokes for about 30 seconds.



### Dry



Leave for approximately 30 seconds, allowing the area to air dry **completely** before proceeding.

Discard the applicator after a single use.